Description de l'appareil

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Tel: +90 (232) 469 14 44

www.wingmed.com.tr - info@wingmed.com.tr

Doğam Medikal company manufacturer of orthopedic equipment under Wingmed brand declares that, marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. Conformity assessment was carried out in accordance with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

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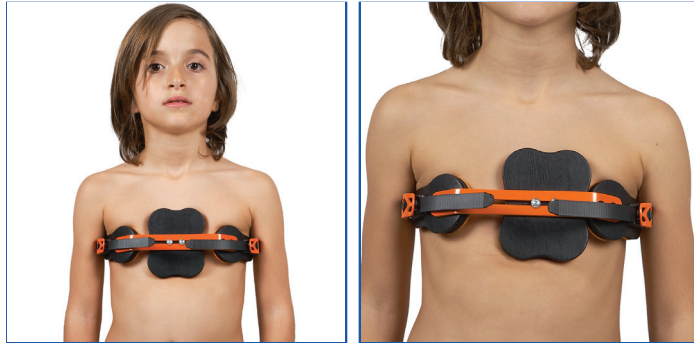
TR

Device Description

It is a type of orthosis used for the treatment of patients with pigeon chest deformity. It prevents the progression of the deformity and helps the treatment by applying appropriate pressure to the protruding part in the chest area thanks to the supports in the front and the adjustable plastic buckles on the sides. The device is designed only for current indications and accurate body measurement.

Device Content

Textile Components: Poliamid-Polyester

Rigid Components: Eva-Aluminum-Steel-Polypropylene

Device Application

After loosening one of the plastic buckles on the sides and placing it on the protrusion formed in the chest area of the pad support in front of the device, place the plastic buckle back in its slot and tighten it appropriately. To ensure the correct tightening motion, the plastic buckles are moved backwards, and when you want to loosen them, the button located in the middle of the plastic buckle must be pressed. With the help of the screws on the pad support in the front, the rotation of the pad support can be achieved.

Indications

- Supporting the chest wall
- In rehabilitation after surgical intervention

Contraindication

- Do not use the product in cases where the diagnosis is not certain.
- Do not contact the product directly with the injured skin.
- Do not use in case of allergy to any of the components of the product.
- Do not use in venous and lymphatic conditions.

Important Information & Warnings

Check the integrity of the device before each use.

Do not use the appliance if it is damaged.

It is recommended that a healthcare professional supervise.

This appliance is only suitable for use by one patient and should not be used by another person.

In case of discomfort, pain, abnormal volume and discoloration of the limb, remove the device and consult a healthcare professional.

Daily use and duration of application should be specified by the healthcare professional.

Oil-containing substances (ointments, creams, lotions) should not be brought into contact.

Device Maintenance & Storage

It should be washed in warm water and by hand with mild detergent.

Store the appliance cleanly in its original box when not in use.

EN

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DOCUMENT REVISIONS 202301(2023-01) - WINGMED ©