Description du dispositif FR

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Device Description EN

This device is usually designed to relieve low back pain, back pain, muscle strains, and pain in this area. The device is designed only for current indications and accurate body measurement.

Device Content Textile Components: Polyamide-Elastane-Polyester
Rigid Components: Steel-Polypropylene

Device Application Place the belt around the waist and close the velcro straps on the front so that they overlap. The extra belts on the sides are fixed by means of velcro to fit the corset perfectly and comfortably on the body.

Indications

- Lumbar vertebral hernias and calcifications
- Sacralization and lumbalization
- Sudden and long-term low back pain
- In rehabilitation after surgical intervention

Contraindications

- Do not use the product in cases where the diagnosis is not certain.
- Do not contact the product directly with the injured skin.
- Do not use in case of allergy to any of the components of the product.
- Venous and lymphatic conditions.

Important Information & Warnings

- Check the integrity of the device before each use.
- Do not use if the device is damaged.
- It is recommended that a healthcare professional supervise.
- Do not use the device while driving.
- This device is only suitable for use by one patient and should not be used by another person.
- In case of discomfort, pain, abnormal volume, and discoloration of the limb, remove the device and consult a healthcare professional.
- The duration of daily use and application should be indicated by the healthcare professional.
- Substances containing oil (ointments, creams, lotions) should not come into contact with the device.

Device Maintenance & Storage

- It should be washed in warm water and hands with mild detergent.
- To preserve the long-term adhesive strength of the velcro tapes, the velcro tapes should be closed before washing.
- It should be dried in the open air without hanging and should not be exposed to heat (sun, heating) when drying.
- Do not dry clean.
- Keep the device clean in its original box when not in use.

Size

BEL ÇEVRESİ
WAIST CIRCUMFERENCE
TOUR DE TAILLE
ОБХВАТ ТАЛИИ(см)

S 70-80 | M 80-90 | L 90-100 | XL 100-110 | XXL 110-120 | XXXL 120-130

STD 1= S - M 70-90 | STD 2= L - XL - XXL 90-120