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**Device Description**

This device is a type of orthosis designed to provide stabilization to the neck area. The device is intended solely for the listed medical indications and is based on accurate body measurements

**Device Content**

Textile Components: Polyester-Polyamide  
Rigid Components: Plastazote, Polypropylene

**Device Application**

Place the neck brace so that it aligns with the chin, then secure the Velcro straps at the front.

**Indications**

- Soft tissue injuries
- Stable bone and ligament injuries
- Postoperative protection and stabilization
- Cervical disc irritation / cervical disc herniation
- Suspected cases of cervical spine trauma in unconscious patients

**Contraindications**

- Do not use the device if the diagnosis is uncertain.
- Do not allow direct contact of the device with injured skin.
- Do not use if allergic to any of the components.
- Do not use in cases of venous or lymphatic disorders.

**Important Information & Warnings**

Inspect the device for damage before each use.  
Do not use if damaged.  
Use under the supervision of a healthcare professional.  
Do not use the device while driving.  
This device is for single-patient use only and should not be used by another person.  
In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional.  
The duration and frequency of daily use should be determined by a healthcare professional.  
Avoid contact with oil-based substances such as ointments, creams, and lotions.

**Device Maintenance & Storage**

Hand wash in warm water using mild detergent.  
To preserve the long-term adhesive strength of the Velcro tapes, close them before washing.  
Air dry flat in an open space without hanging and avoid exposure to heat sources.  
Do not dry clean.  
Keep the device clean and stored in its original box when not in use.