COLLIER CERVICAL RIGIDE AVEC MENTONNIERE
RIGID CERVICAL COLLAR WITH CHIN SUPPORT

REF W107

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Device Description

This device is a type of orthosis designed to provide stabilization to the neck area. The device is intended solely for the listed medical indications and is based on accurate body measurements

Device Content

Textile Components: Polyester, Polyamide, Polyurethane (PU)
Rigid Components: Polyethylene

Device Application

Position the cervical collar so that it fits under the chin, then fasten the Velcro strap located at the back. Height adjustment can be made using the side Velcro strap.

Indications

- Cervical disc herniation
- Cervical osteoarthritis
- Postoperative protection and stabilization

Contraindications

- Do not use the device if the diagnosis is uncertain.
- Do not allow direct contact of the device with injured skin.
- Do not use if allergic to any of the components.
- Do not use in cases of venous or lymphatic disorders.

Important Information & Warnings

Inspect the device for damage before each use. Do not use if damaged. Use under the supervision of a healthcare professional. Do not use the device while driving. This device is for single-patient use only and should not be used by another person. In case of discomfort, pain, abnormal swelling or discoloration of the skin, remove the device and consult a healthcare professional. The duration and frequency of daily use should be determined by a healthcare professional. Avoid contact with oil-based substances such as ointments, creams, and lotions.

Device Maintenance & Storage

Hand wash in warm water using mild detergent. To preserve the long-term adhesive strength of the Velcro tapes, close them before washing. Air dry flat in an open space without hanging and avoid exposure to heat sources. Do not dry clean. Keep the device clean and stored in its original box when not in use.