Description de l'appareil

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Tel: +90 (232) 469 14 44

www.wingmed.com.tr - info@wingmed.com.tr

Doğam Medikal company manufacturer of othopedic equipment under Wingmed brand declares that, marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. Conformity assessment was carried out in accordance with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

tr	Medikal Cihaz	Tek hasta Çoklu Kullanım	Talimat kitapçığını okuyun
en	Medical Device	Single patient Multiple Use	Read the instruction leaflet
fr	Dispositif médical	Un seul patient à usage multiple	Lire attentivement la notice
ru	Медицинский прибор	Многократное использование для одного пациента	Внимательно прочтите инструкцию

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TR

Device Description

This device provides immobilization of the shoulder at 45 or 60 degree abduction. The device is designed only according to the available indications and accurate body measurement.

Device Content

Textile Components: Polyester-Polyamide-PU Sponge

Rigid Components: Polypropylene

Device Application

Installation of this product requires the help of a person. First, secure the pillow with the help of velcro tape so that it is at waist level. Your arm After placing it on the arm strap, you pass the strap from the other shoulder to the front Combine with the buckle. By changing the position of the pillow, you can apply the abduction angle as 45 or 60 degrees.

Indications

- Rotator cuff muscle tears
- Shoulder calcifications
- Postoperative immobilization
- In rehabilitation after surgical intervention
- In rehabilitation after frozen shoulder arthroscopy

Contraindication

- Do not use the product in cases where the diagnosis is not certain.
- Do not contact the product directly with the injured skin.
- Do not use in case of allergy to any of the components of the product.
- Do not use in venous and lymphatic conditions.

Important Information & Warnings

Check the integrity of the device before each use. Do not use if the device is damaged. It is recommended that a healthcare professional supervise. Do not use the Device while driving. This device is only suitable for use by one patient and should not be used by another person. In case of discomfort, pain, abnormal volume and discoloration of the limb, remove the device and consult a healthcare professional. The duration of daily use and application should be indicated by the health care professional. Substances containing oil should not be brought into contact with (ointments, creams, lotions). **Device Maintenance & Storage** It should be washed in warm water and hands with mild detergent. To preserve the long-term adhesive strength of the velcro tapes, the velcro tapes should be closed before washing. It should be dried in the open air without hanging and should not be exposed to heat (sun, heating) when drying. Do not dry clean. Keep the device clean in its original box when not in use.

EN