Description de l'appareil

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Tel: +90 (232) 469 14 44

www.wingmed.com.tr - info@wingmed.com.tr

Doğam Medikal company manufacturer of orthopedic equipment under Wingmed brand declares that, marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. Conformity assessment was carried out in accordance with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

tr	Medikal Cihaz	Tek hasta Çoklu Kullanım	Talimat kitapçığını okuyun
en	Medical Device	Single patient Multiple Use	Read the instruction leaflet
fr	Dispositif médical	Un seul patient à usage multiple	Lire attentivement la notice
ru	Медицинский прибор	Многократное использование для одного пациента	Внимательно прочтите инструкцию

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TR

Device Description

It is used to take a resting position after sprains of the thumb and wrist, strains and casts. It ensures that the thumb and wrist are kept immobile in a neutral position after the surgery. The device is designed only for current indications and accurate body measurement.

Device Content

Textile Components: Polyester-Polyamide-PU Sponge

Rigid Components: Aluminum

Device Application

After opening the velcro straps on the device, pass your thumb through the slot and fix the velcro towards the back, starting from the velcro on the front. After checking that the position of your wrist is suitable, tightening and loosening can be done with the help of velcro. The device is packaged for right-handed use from the emble, while for left-handed use it is possible by replacing the aluminum support on the bottom with a slot on the top.

Indications

- In de quervain syndrome
- Carpal Tunnel Syndrome
- Tendon and soft tissue damage
- In rehabilitation after surgical intervention

Contraindication

- Do not use the product in cases where the diagnosis is not certain.
- Do not contact the product directly with the injured skin.
- Do not use in case of allergy to any of the components of the product.
- Do not use in venous and lymphatic conditions.

Important Information & Warnings

Check the integrity of the device before each use. Do not use if the device is damaged. It is recommended that a healthcare professional supervise. Do not use the Device while driving. This device is only suitable for use by one patient and should not be used by another person. In case of discomfort, pain, abnormal volume and discoloration of the limb, remove the device and consult a healthcare professional. The duration of daily use and application should be indicated by the health care professional. Substances containing oil should not be brought into contact with (ointments, creams, lotions).

Device Maintenance & Storage

It should be washed in warm water and hands with mild detergent. To preserve the long-term adhesive strength of the velcro tapes, the velcro tapes should be closed before washing. It should be dried in the open air without hanging and should not be exposed to heat (sun, heating) when drying. Do not dry clean. Keep the device clean in its original box when not in use.

EN