Description de l'appareil

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Tel: +90 (232) 469 14 44

www.wingmed.com.tr - info@wingmed.com.tr

Doğam Medikal company manufacturer of othopedic equipment under Wingmed brand declares that, marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. Conformity assessment was carried out in accordance with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

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Device Description

This device is designed to passively extend, allowing for active finger flexion by passively holding the wrist and fingers. The device is designed only according to the available indications and accurate body measurement.

Device Content

Textile Components: Polyester-Polyamide-PU Sponge

Rigid Components: Polypropylene-Aluminum-Steel

Device Application

Place the orthosis so that it is on top of your hand. Adjust the straps in turn to achieve a tight and comfortable fit. Place your fingers on the fabrics located at the ends of the springs and comfortably adjust them to yourself.

Indications

- In correcting muscle immobility
- Ulnar nerve injuries
- Extensor tendon injuries
- In rehabilitation after surgical intervention

Contraindication

- Do not use the product in cases where the diagnosis is not certain.
- Do not contact the product directly with the injured skin.
- Do not use in case of allergy to any of the components of the product.
- Do not use in venous and lymphatic conditions.

Important Information & Warnings

Check the integrity of the device before each use. Do not use if the device is damaged. It is recommended that a healthcare professional supervise. Do not use the Device while driving. This device is only suitable for use by one patient and should not be used by another person. In case of discomfort, pain, abnormal volume and discoloration of the limb, remove the device and consult a healthcare professional. The duration of daily use and application should be indicated by the health care professional. Substances containing oil should not be brought into contact with (ointments, creams, lotions).

Device Maintenance & Storage

It should be washed in warm water and hands with mild detergent. To preserve the long-term adhesive strength of the velcro tapes, the velcro tapes should be closed before washing. It should be dried in the open air without hanging and should not be exposed to heat (sun, heating) when drying. Do not dry clean. Keep the device clean in its original box when not in use.

UNIVERSAL
SIZE

Size

BİLEK ÇEVRESİ
WRIST CIRCUMFERENCE
TOUR DU POIGNET
ОКРУЖНОСТЬ ЗАПЯСТЬЯ(см)

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DOCUMENT REVISIONS 202301(2023-01) - WINGMED ©