Description du dispositif

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Tel: +90 (232) 469 14 44

www.wingmed.com.tr - info@wingmed.com.tr

Dogam Medikal company manufacturer of orthopedic equipment under Wingmed brand declares that, marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. Conformity assessment was carried out in accordance with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

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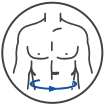
TR

Device Description

It is used in fractures in the spine, hernias, slips, situations where the body needs to stand upright and to eliminate posture disorders. The device is designed only for current indications and accurate body measurement.

Device Content

Textile Components: Polyamide-Elastane-Polyester

Rigid Components: Steel-Polypropylene

Device Application

Place the belt around the waist and close the velcro straps on the front so that they overlap. The extra belts coming from the shoulder are crossed from the back and pulled to the front. The extra belts pulled forward are fixed by means of velcro to fit the corset perfectly and comfortably to the body.

Indications

- In all vertebral hernias
- Sudden and long-term low back pain
- In osteoporosis
- In rehabilitation after surgical intervention

Contraindication

- Do not use the product in cases where the diagnosis is not certain.
- Do not contact the product directly with the injured skin.

- Do not use in case of allergy to any of the components of the product.
- Do not use in venous and lymphatic conditions.

Important Information & Warnings

Check the integrity of the device before each use. Do not use if the device is damaged. It is recommended that a healthcare professional supervise. Do not use the Device while driving. This device is only suitable for use by one patient and should not be used by another person. In case of discomfort, pain, abnormal volume and discoloration of the limb, remove the device and consult a healthcare professional. The duration of daily use and application should be indicated by the health care professional. Substances containing oil should not be brought into contact with (ointments, creams, lotions).

Device Maintenance & Storage

It should be washed in warm water and hands with mild detergent. To preserve the long-term adhesive strength of the velcro tapes, the velcro tapes should be closed before washing. It should be dried in the open air without hanging and should not be exposed to heat (sun, heating) when drying. Do not dry clean. Keep the device clean in its original box when not in use.

EN

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DOCUMENT REVISIONS 202407(22/07/2024) - WINGMED ©