Description de l'appareil

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Tel: +90 (232) 469 14 44

www.wingmed.com.tr - info@wingmed.com.tr

Dogam Medikal company manufacturer of othopedic equipment under Wingmed brand declares that, marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. Conformity assessment was carried out in accordance with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council.

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TR

Device Description

This device is an ankle splint that provides stabilization of the joint. The device is designed only according to the available indications and accurate body measurement.

Device Content

Textile Components: Polyester-Polyamide-PU Sponge

Rigid Components: Polypropylene

Device Application

The device should be worn in a seated position. Loosen the straps and put your foot on two plastic supports and adjust the width using the velcro located under the heel. Tighten the lower strap and then tighten the upper strap, which can be used either with shoes or outside. Gently slide the foot into the shoe along with the device.

Indications

- Chronic ankle imbalance
- Moderate to severe ankle sprain
- Ligament damage
- In rehabilitation after surgical intervention

Contraindication

- Do not use the product when the diagnosis is uncertain.
- Do not contact the product directly with injured skin.
- Do not use in case of allergy to any of the product components.
- Do not use in cases of Venous and Lymphatic.

Important Information & Warnings

Check the integrity of the device before each use. Do not use if the device is damaged. It is recommended that a healthcare professional supervise. Do not use the Device while driving. This device is only suitable for use by one patient and should not be used by another person. In case of discomfort, pain, abnormal volume and discoloration of the limb, remove the device and consult a healthcare professional. The duration of daily use and application should be indicated by the health care professional. Substances containing oil should not be brought into contact with (ointments, creams, lotions).

Device Maintenance & Storage

It should be washed in warm water and hands with mild detergent. To preserve the long-term adhesive strength of the velcro tapes, the velcro tapes should be closed before washing. It should be dried in the open air without hanging and should not be exposed to heat (sun, heating) when drying. Do not dry clean. Keep the device clean in its original box when not in use.

EN

Size

UNIVERSAL
SIZE

BİLEK ÇEVRESİ
ANKLE CIRCUMFERENCE
TOUR DE LA CHEVILLE
ОКРУЖНОСТЬ ЛОДЫЖКИ(cm)

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DOCUMENT REVISIONS 202301(2023-01) - WINGMED ©